




INSTALLATION INSTRUCTIONS

FLIP UP Shower Seat SMFSS 1313 seat

Questions, problems, missing parts?
Before returning to your dealer or
retailer, Please call our customer ser-
vice at 1-844-974-2400, 8 am - 5 p.m.
EST, Monday - Friday

Please read these instructions thoroughly before
installing this shower seat.

**IMPORTANT: THIS PRODUCT SHOULD ONLY BE INSTALLED BY QUALIFIED AND
TRAINED PERSONNEL**

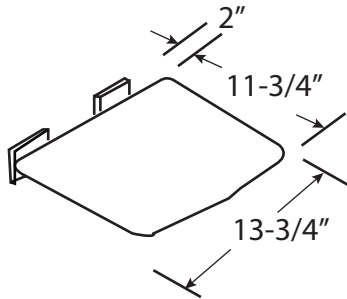

FAILURE TO FOLLOW THESE INSTRUCTIONS COULD LEAD TO IMPROPER INSTALLATION OF THE SHOWER SEAT, STRUCTURAL AND/OR WATER DAMAGE TO THE WALL AND IN-WALL SUPPORT STRUCTURES, AND/OR SERIOUS BODILY INJURY IN USE. IN ADDITION, ALL WARRANTIES, EXPRESS OR IMPLIED, WILL BE NULL & VOID DUE TO THE IMPROPER INSTALLATION OF THIS PRODUCT. THE MANUFACTURER DOES NOT ASSUME ANY LIABILITY FOR DAMAGE RESULTING FROM SERVICES PERFORMED BY OTHERS, OR FAULTY INSTALLATION, MISUSE OR MISAPPLICATION OF GOODS. TO ASSURE PROPER AND SAFE INSTALLATION, THIS SHOWER SEAT **MUST BE INSTALLED INTO A MINIMUM 2" THICK SOLID WOOD IN-WALL BACKING MATERIAL.** MAKE CERTAIN IN-WALL BACKING COMPLIES WITH ALL LOCAL BUILDING CODE REQUIREMENTS. PLEASE ADHERE TO MAXIMUM WEIGHT CAPACITY OF 330 LBS. (150 KGS.)

Hardware Provided: (8) M5 x 60mm Phillips Head Screws

Tools Required: Marking Pencil, level, power drill, 3/8" carbide tip drill bit, #2 Phillips driver, silicone

STEP 1: LOCATING your SHOWER SEAT

- Remove the Shower Seat from package.
- Locate the area that contains the 2" solid wood in-wall backing on the desired mounting surface.
- Place the mounting bracket in position on the wall. Using a level (not provided), ensure the mounting hinges are level and pencil-mark the location of the mounting holes on the wall.
- Using a pencil or suitable marker, it is extremely important you mark the center of each of the mounting holes precisely on the mounting surface.

PROCEED CAUTIOUSLY through the following steps in this installation to avoid drilling into in-wall water supply lines. **ALL MOUNTING POINTS MUST FASTEN INTO STRUCTURAL WOOD BACKING, with a minimum 2" thickness. Avoid electrical and/or plumbing lines behind the wall. Where possible drill through tile and do not drill through grout.**

STEP 2: MOUNTING your SHOWER SEAT

Figure 2.1

Using a power drill with a 3/16" carbide tip drill bit (not provided), drill the eight (8) locations marked on the wall in the previous step 1. Apply a bead of silicone inside each of the pre-drilled holes to ensure a water-tight seal will be created at all mounting locations.

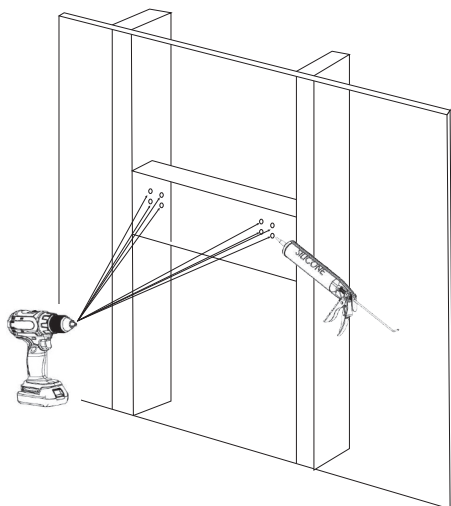


Figure 2.1

STEP 2: MOUNTING your SHOWER SEAT

Figure 2.2

Apply a bead of silicone around the entire edge of the Shower Hinges at the rear of the Shower Seat. Secure the Shower Seat to the wall using the supplied screws by screwing them into the pre-drilled and silicone filled holes. Please note maximum weight capacity when properly installed with a 2" wood in-wall backer is 330 lbs.

Figure 2.2

